# **USB WRIST BLOOD PRESSURE MONITOR**

**QUICKSTART GUIDE**  
ENGLISH ( 3 – 21 )

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# BEFORE YOU START

## WARNINGS AND PRECAUTIONS

- Do NOT use this device on infants or persons who cannot communicate. Doing so may lead to serious accidents.
- This device does not serve as a cure of any symptoms or disease. The data measured are only for reference.
- Always consult your physician to have the results interpreted.
- Use of this device near a mobile phone or microwave oven may cause inaccurate results.
- Do not use this device for purposes other than measuring blood pressure.
- Disassembly or modification the unit or wrist cuff may lead to permanent reading errors.
- This device is not able to measure in the presence of common arrhythmia, such as atrial or ventricular premature beats or atrial fibrillation. It may produce reading errors.

## **INTENDED USE**

Thank you for purchasing the ION Health USB Blood Pressure Monitor. This device measures blood pressure non-invasively based on the oscillometric method. This device is designed for individuals age 18 and above for home use only. Please read these instructions carefully before using the device. For other questions regarding this monitor, please contact ION Health.


Please keep your receipt after purchasing this product. To register your product online, please visit our website: <http://www.ionhealthonline.com>

### **Features:**

- One-touch, accurate results in 20 seconds.
- Comfortable automatic inflation and deflation.
- Memories for readings up to 352 sets, with time/date and automatically calculated average.
- Automatic power-off after 3 minutes of no activity.

## HEALTH INFORMATION

Knowing your blood pressure trend can help you tell whether your body is in good condition or not. Human blood pressure naturally increases after reaching middle age. This symptom is a result of continuous aging of blood vessels. Further causes include obesity, lack of exercise, and cholesterol (LDL) adhering to the blood vessels. Rising of blood pressure accelerates hardening of the arteries, and the body becomes more susceptible to apoplexy and coronary infarction. The World Health Organization (WHO) has published the following guideline of blood pressure range: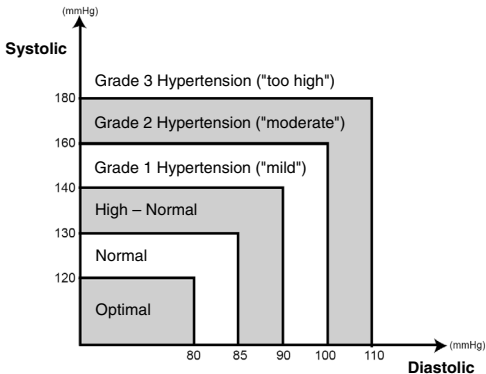


## **ABOUT THE DEVICE**

### **Box Contents:**

- Blood pressure monitor
- Wrist cuff
- USB cable
- Software CD
- Storage case
- 2 AAA batteries
- Quickstart Guide

Please check the box to ensure you have everything listed above. If not, please return the product where you purchased it.

## OVERVIEW

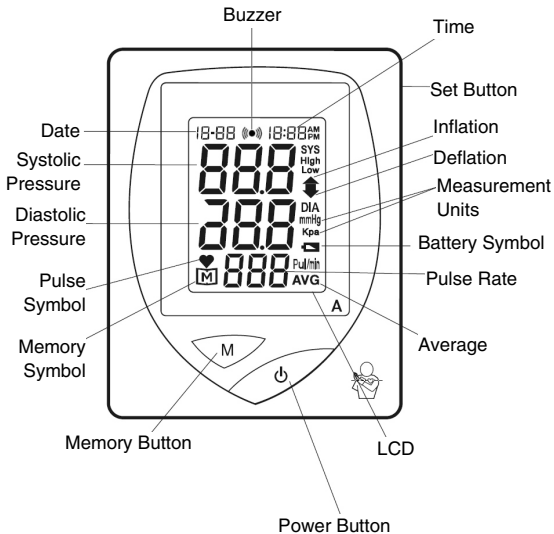## BEFORE USE

Your monitor comes with batteries already installed. When replacing them, use ONLY 1.5V AAA size alkaline batteries. Please use alkaline batteries for the best performance and the longest life.

**Low Power Warning:** The monitor will remind you when the power is getting low by displaying two different messages:

- **The Battery Symbol appears with your test result.** This means it is nearly time to change the batteries (although still 30 more measurements can be made).
- **The Battery Symbol flashes by itself.** This means you must change the batteries before making any measurements.

## **Batteries Installation and Replacement:**


1. Please make sure the device is turned off. Press the edge of the battery cover and push upward.

**Note:** *Replacing the batteries does not affect previous test results stored in the device, but you may need to update the settings.*

2. Remove the battery cover then remove the batteries inside.

**Note:** *Do not use different type, date or brand name batteries together. Use only new batteries of the required size and type.*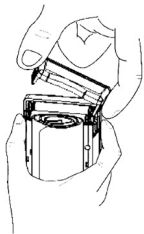


3. Insert two 1.5V AAA size alkaline batteries. Make sure the "+" (positive) and "-" (negative) ends match those indicated in the battery compartment. If inserted correctly, you will hear a "beep."

### **Setting the Device:**

Your monitor comes with the time, date, and measurement units preset. These options may be changed any time to fit your needs. They also need to be set again after you change the batteries.

1. With the monitor turned off, remove the battery cover as shown in previous page. Press the Set Button located in the battery compartment to enter Setting Mode.
2. Press the Set Button to select what you would like to change. The selected one will be flashing. Press the Memory Button to make change.
3. Save the Changes by pressing the Set Button and move on to the next step.

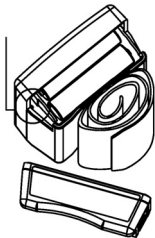**Example:**

1. With the device turned off, remove the battery cover as shown in previous page. Press the Set Button located in the battery compartment to enter Setting Mode. The year will appear first, with the number flashing.
2. Press and release the Memory Button to advance one year.

**Note:** To move the number faster, hold the Memory Button down.

3. Press the Set Button to set the year. The date will appear on the display with the month flashing.
4. Repeat Steps 2 – 3 to set the month, day, hour, minute, measurement units (in mmHg or KPa), and buzzer (on/off).
5. After choosing the buzzer setting (on/off), press the Set Button to save the settings and turn off the device.

**Note:** You may also press the Power Button any time to turn off the device and exit the Setting Mode.

# USING THE DEVICE

## MEASURING YOUR BLOOD PRESSURE

### Suggestions before Measuring:

- Avoid caffeine, tea, alcohol, and cigarettes at least 30 minutes before measurement.
- Wait 30 minutes after exercising or bathing before measurement.
- Sit or lie down for at least 10 minutes before measuring.
- Do not measure when feeling anxious or tense.
- Please take a 5-10 minute break between measurements.
- Waiting allows the engorged blood vessels to return to normal status. This break can be longer if necessary, depending on your physical conditions.
- Keep the records for your physician as reference.


### **Applying the Wrist Cuff:**

1. Hold your left arm in front of you with your palm facing up. Slide the cuff onto your wrist. Make sure the monitor is on the inside of your wrist with the display in easy view.

2. Wrap the cuff comfortably around your wrist. The top and bottom edges of the cuff should be tightened evenly around your wrist.

**Note:** The top margin of the monitor should be about 1/4 to 1/2 inch (0.6 cm to 1.3 cm) below the ball of thumb.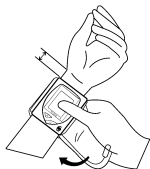

3. When the cuff is positioned correctly, fold the loose end back and secure it in place by pressing the pile material firmly against the hook material.

**Note:** Blood pressure varies from two hands naturally. Always measure your blood pressure on the same wrist.

## Testing Your Blood Pressure:

1. Sit or lie down for at least 10 minutes before measuring. Take the measurement while sitting.
2. Apply the wrist cuff (as described earlier).
3. Press the **Power Button** and place your arm in front of your chest. Relax and make sure the device is at the same height as your heart. Holding your left elbow with your right hand helps you hold still during measurement. Remain still and do not talk or move during the measurement.

4. After turning on the device, the cuff automatically inflates and **Inflation Symbol** appears. You will see the number increasing on the display.

After the cuff pressure is reached, the cuff begins to deflate. As it deflates, the **Deflation Symbol** will appear on the display. You will also see the number decreasing and **Pulse Symbol** flashing on the display when a pulse is detected.

### Testing Your Blood Pressure (continued):

5. After completely deflation, the monitor displays the systolic pressure, diastolic pressure, and heart rate.
6. Press the **Power Button** to turn off the device.

### **Notes:**



- It is extremely important that the cuff is at the same level of the heart. If the wrist is relatively lower or higher than the heart, the blood pressure measured will be higher or lower (respectively) than the actual value.
- If a higher pressure value is needed, the monitor will stop deflation and inflate again.
- If you press the **Power Button** during the measurement, the monitor will turn off.

## MEMORY

Your monitor stores the most recent 352 sets of readings. You can review them in the monitor itself or in your personal computer. (Make sure your computer meets the minimum system requirements for ION Health Suite.)

### Viewing Results on the Monitor:

1. When the monitor is off, press the **Memory Button**. You will see the average of all data stored in the device, including systolic and diastolic blood pressure, and pulse.
2. Press the **Memory Button** again. The number **1** will appear first, followed by the latest reading.
3. Press the **Memory Button** again to view the other results in the order of time and date taken. After viewing the oldest test result, push down the **Memory Button** again to resume to the latest test result. When the memory is full, the oldest results are deleted as new ones are added.
4. To exit the memory anytime, press the **Power Button** to turn off the monitor.

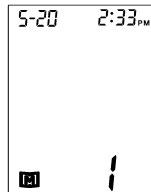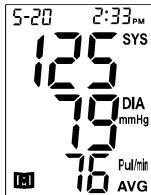
### **Viewing Results on a Personal Computer:**

Results in the device's memory can be transferred to your computer and viewed/stored in the included ION Health Suite software.

1. Insert the included software CD into your computer's CD-ROM drive and follow the on-screen instructions to install ION Health Suite.
2. Connect the included USB cable to a free USB port on the back of your computer. With the device turned off, connect one end of the USB cable to the USB port on the side of the monitor. **PCL** ("PC Link") will appear on the display, indicating that the device is ready to transfer your results to your computer.
3. Follow the instructions provided in the software to transfer your results. The results will include their dates and times. When finished, disconnect the cable. (The device will automatically turn off.)

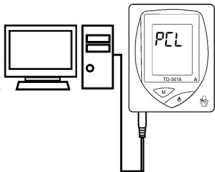**Note:** While the device is connected to your computer, it cannot measure your blood pressure.

## Erasing the Memory:

**Note:** Be sure to turn the device off before erasing all readings.

1. Press and hold the **Memory Button** for five seconds until **CLr ALL** appears on the display.

2. Release the **Memory Button** and the device will automatically turn off. Its memory will be erased.

**Note:** When there are no test results in memory, you will only see the date, time, and the Memory Symbol.

## MAINTENANCE

### Storage

- Protect the device from direct sunlight and humidity. Its performance specifications could be adversely affected if stored or used beyond the specified temperature and humidity ranges listed below:
  - **Operating conditions:** 10°C – 40°C (50°F – 104°F), 30 – 85% relative humidity
  - **Storage conditions:** -20°C – 60° C (-4°F – 140°F), 10 – 95% relative humidity
- Do not drop, shake, or hit the device.
- Disassembly or modification the unit or wrist cuff may lead to permanent reading error.
- Always use the included storage case to transport or store it.
- Batteries may leak chemicals that damage electronic parts. Please remove the batteries if you are not going to use the device for an extended period (i.e., 3 months or more).

### Cleaning

- To clean the unit, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry it with a soft, dry cloth. Do not flush with or immerse it in water.
- Do not use benzene, gasoline, or any alcohol-based or solvent agent to clean the device.
- To clean the cuff, wipe it with a moistened cloth and soap. Do not immerse it in water.

## **Cleaning (continued)**

- Do not twist any of the device's tubing. Do not stretch, bend, or tug on the wrist cuff. Do not turn the cuff inside-out.
- Do not separate the cuff from the unit, disassemble the device, or try to repair the unit yourself. If any problem occurs, contact ION Health for service.

## **TROUBLESHOOTING**

Please check the chart below for problems you can fix on your own (no tools required). For error messages other than the ones below, do not try to repair by yourself. Please contact ION Health for service.

<b>PROBLEM</b>	<b>CAUSE</b>	<b>SOLUTION</b>
The display shows "Err 00."	Weak pulse.	Refit the cuff more snugly, relax, and repeat the measurement.
The display shows "Err 02."	Systolic pressure cannot be detected.	Refit the cuff more snugly, relax, and repeat the measurement.
The display shows "Err 04."	Diastolic pressure cannot be detected.	Refit the cuff more snugly, relax, and repeat the measurement.
The display shows "Err 09."	Cuff pressure is not enough even at the highest amount.	Contact ION Health for service.

<b>PROBLEM</b>	<b>CAUSE</b>	<b>SOLUTION</b>
The display is blank after pressing the Power Button.	Batteries are not installed properly.	Ensure the batteries' "+" (positive) and "-" (negative) ends are properly oriented.
	Batteries are low on power.	Replace them with new batteries.
Battery Symbol appears on display.	Batteries were stored at low temperatures (which causes them to perform poorly).	Let the batteries warm up to room temperature (but do not directly heat them) before using them.
	Batteries are low on power.	Replace them with new batteries.
Result is higher or lower than my average measurements.	You may not be taking measurements under ideal physical conditions.	See the "Suggestions before measuring" notes under MEASURING BLOOD PRESSURE.
Heart rate is higher or lower than my average measurement.	You may not be taking measurements under ideal physical conditions.	See the "Suggestions before measuring" notes under MEASURING BLOOD PRESSURE.
The device turns off automatically after a few minutes.	This is an intended feature to prevent wasting battery power.	Power on the device by pressing the Power Button.

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## SPECIFICATIONS

<b>Display</b>	LCD digital display
<b>Measurement Method</b>	Oscillometric
<b>Measurement Range</b>	Pressure: 0~300 mmHg Pulse rate: 40~199 / min.
<b>Accuracy/Calibration</b>	Pressure: $\pm 3$ mmHg or $\pm 2\%$ of reading Pulse rate: $\pm 4\%$ of reading
<b>Memory Capacity</b>	352 sets of readings with date and time
<b>Inflation</b>	Automatic
<b>Deflation</b>	Electric valve
<b>Pressure Release</b>	Automatic exhaust valve
<b>Pressure Detection</b>	Piezo-resistive silicon pressure transducer
<b>Power Supply</b>	2 AAA 1.5v alkaline batteries
<b>Operation Environment</b>	10° C – 40° C (50°F – 104°F) 30 ~ 85% R.H. maximum
<b>Storage Environment</b>	-20° C – 60° C (-4°F – 140°F) 10 ~ 95% R.H. maximum
<b>Battery Life</b>	Approx. 200 uses
<b>Cuff Size</b>	Wrist circumference approx. 135~195mm (5.25" ~ 7.75")
<b>Dimensions</b>	76mm x 64mm x 29mm
<b>Weight</b>	132g (with batteries)
<b>Accessories</b>	USB cable, software CD, storage case, 2 AAA batteries, Quickstart Guide

**WARRANTY INFORMATION:** Warranty and technical support information can be found online at [www.ionhealthonline.com](http://www.ionhealthonline.com) or by calling: **877-466-3170** (U.S.), **01252 896 080** (U.K.).

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SYMBOL	REFERENT
	Read instructions before use
	Manufacturer
	Serial number
	Caution: Consult accompanying documents
	Authorized representative in the European Community
	Do not use if package is damaged
	Type BF equipment
	CE mark

## Distributed by ION Health

### **U.S. Office:**

200 Scenic View Dr.  
Cumberland, RI 02864  
United States

### **U.S. Customer Care:**

877-ION-3170 (877-466-3170)  
Monday – Friday, 8:30 am – 6:30 pm (EST)

### **U.K. Office:**

Unit 3, Nexus Park  
Lysons Avenue  
Ash Vale  
Hampshire  
GU12 5QE  
United Kingdom

### **U.K. Customer Care:**

01252 896 080  
Monday – Thursday, 9:00 am – 6:00 pm (GMT)  
Friday, 9:00 am – 5:00 pm (GMT)

MedNet GmbH  
Borkstrasse 10, 48163 Muenster – Germany  
Tel: +49 251 322660  
Fax: +49 251 3226622

TaiDoc Technology Corporation  
3F, 5F, No. 127, Wugong 2nd Road, 24888  
Wugu Dist., New Taipei City, Taiwan

# [www.ionhealthonline.com](http://www.ionhealthonline.com)

**Read instructions before use.**

MANUAL VERSION 1.0  
2011.08.31